

Richard G. Fried, M.D., Ph.D.
Fern G. Fried, M.D.
Priya B. Dhanaraj, M.D.
Judith A. Cenci, M.D.
Carmen Campanelli, M.D.



Lauren Sternberg, M.D.
Roman Bronfenbrener, M.D.
Amy Matorin, PA-C
R. Francesca Wolfe, PA-C

MEDIHONEY
Calcium Alginate Dressing

Directions for use:

- Prior to application of MEDIHONEY Calcium Alginate, cleanse the wound area as necessary with saline solution, soapy water, or hydrogen peroxide.
- Use an appropriately sized MEDIHONEY Calcium Alginate dressing. Cut to size, slightly overlapping the wound area by about ½ inch.
- Use an appropriate secondary dressing to cover the MEDIHONEY Calcium Alginate (waterproof bandage).
- Change bandage every 2-3 days.
- Rinse & clean wound only between bandage changes.

If the dressing is not easily removed, soak with sterile saline or soapy water until it is removed without difficulty.